

dNote™ Driver Pack Version 3.5

This document includes release note information for Version 3.5 of the dNote Driver Pack.

IMPORTANT NOTE: Driver Pack 3.5 can only be installed against UniPOC 1.6.4 and higher or UniPOC 1.7.4 and higher, and the latest release/patch of dNote 3.2 or higher. Attempting to install against any other versions will result in the installer package programmatically cancelling the install attempt.

Unidirectional Device Driver Support via Meditrac

- *Patient Results*
- *QC Results*
- *Calibration Results*
- *Proficiency Results (introduced with Driver Pack 2.5 – device type specific)*
- *Linearity Results (introduced with Driver Pack 2.5 – device type specific)*

Bidirectional Device Driver Support via Meditrac

- *Patient Lists*
- *Operator Lists*
- *Location Lists*
- *Reagent Lists*
- *QC Lot Lists*
- *Configurations per Device Type*
- *Comment Lists (introduced with Driver Pack 2.5)*

Please consult the **UniPOC Driver Functionality Matrix** spreadsheet in the Product Information Portal for specific functionality supported by UniPOC for each device type.

Device Driver Support – Full List

Device Type	Driver Pack Install Name	Meditrac DLP Name	DLP Version ¹
Abaxis Piccolo Xpress	Piccolo	DLP_Piccolo.exe	6.5.5.44013
Abbott POC Afinion AS 100	AfinionAS100	DLP_AfinionAs100.exe	6.4.92.41492
Abbott POC Afinion 2	Afinion2	DLP_AfinionAs100.exe	6.4.92.41492
Abbott POC Alere I	AlereI	DLP_AlereI.exe	6.4.57.41037
Abbott POC Cell Dyn Emerald	Emerald	DLP_CellDynEmerald.exe	6.4.103.41037
Abbott ID Now	IDNow	DLP_IdNow.exe	6.5.22.43312
Analyticon Urilyzer 100 / 100 Pro	Urilyzer100	DLP_Urilyzer100.exe	6.4.41.34951
Analyticon Urilyzer 100 Pro POCT1a	Urilyzer100ProPoct1a	DLP_Urilyzer100ProPOCT1a.exe	6.4.23.39809
BD Veritor	BDVeritor	DLP_BDCloud.exe	6.4.34.41037
Boule Medical Medonic M-series - M16, M20, M16M, M20M, M16C, M20C, M16S, M20S, M32B, M32M, M32C, M32S	Medonic	DLP_Swelab_Alpha.exe	6.4.100.41037
Boule Medical Swelab Alpha - Basic, Plus Basic, Standard, Plus Standard, Cap, Plus Cap, Sampler, Plus Sampler	Medonic	DLP_Swelab_Alpha.exe	6.4.100.41037
Cepheid GeneXpert Omni	GeneXpertOmni	DLP_GenExpert.exe	6.5.43.43890
Cepheid GeneXpert PCR	GeneXpert	DLP_GenExpert.exe	6.5.43.43890
Cepheid GeneXpert Xpress	GeneXpert_Poet1a	DLP_GeneXpert_Poet1a.exe	6.5.29.43099
Chirus Dxpess Reader	DXpress	DLP_DXpress.exe	6.4.29.16746
CoaguSense Coag-Sense	CoagSense	DLP_CoagSensePT2.exe	6.5.12.44012
Druker Diagnostics QBC Star	QBCStar	DLP_QBCStar.exe	6.4.56.41037
Haemonetics TEG 6s	TEG6s	DLP_TEG6sHI7.exe	6.5.19.43312
HemoCue 201DM Glucose	HemoCue	DLP_HemoCue.exe	6.4.0.2
HemoCue 201 DM Glucose RT	HemoCue	DLP_HemoCue.exe	6.4.0.2
HemoCue 201 DM Hemoglobin	HemoCue	DLP_HemoCue.exe	6.4.0.2
LumiraDX	LumiraDX	DLP_LumiraDx.exe	6.5.15.44205
Medtronic ACT Plus	ActPlus	DLP_ActPlus.exe	6.4.100.41037
Medtronic HMS Plus	Hms Plus	DLP_Hms_Plus.exe	6.4.93.40298
Nova Biomedical NovaNet (Test Results & Operator Lists only)	NovaNet-Operator (Operators) NovaNet-Result (Test Results)	DLP_NovaNetOperatorResultUniPOC.exe	6.5.46.44689
Nova Biomedical StatSensor Creatinine		via NovaNet only	
Nova Biomedical StatStrip Glucose Meter		via NovaNet only	
Nova Biomedical StatStrip Lactate		via NovaNet only	
Nova Biomedical StatStrip Ketone		via NovaNet only	
Nova Biomedical Prime Plus		via NovaNet only	
Philips BiliChek	BiliChek	DLP_BiliChek.exe	6.4.7.16746
Quidel Sofia, Sofia 2	Sofia	DLP_Sofia.exe	6.5.25.43762
Quidel Triage Meter Pro	TriageMeterProTmsi	DLP_TriageMeterProTmsi.exe	6.4.109.41435
Radiometer ABL80 Flex	ABL_8xx_H	DLP_ABL_H_FW_2_1.exe	6.5.13.42574
Radiometer ABL80 Flex CO OX	ABL_8xx_H	DLP_ABL_H_FW_2_1.exe	6.5.13.42574
Radiometer ABL80 Flex CO-OX OSM	ABL_8xx_H	DLP_ABL_H_FW_2_1.exe	6.5.13.42574
Radiometer ABL90 FLEX	ABL_90	DLP_ABL_H_FW_2_1.exe	6.5.13.42574
Radiometer ABL90 FLEX Plus	ABL_90	DLP_ABL_H_FW_2_1.exe	6.5.13.42574
Roche Diagnostics AccuChek Inform II	AccuChekII	DLP_AccuChekII.exe	6.4.104.41037
Roche Diagnostics CoaguChek XS Plus	CoaguChekXsPlus	DLP_CoaguChekXsSeries.exe	6.4.80.39284
Roche Diagnostics CoaguChek XS Pro	CoaguChekXsPlus	DLP_CoaguChekXsSeries.exe	6.4.80.39284
Roche Diagnostics CoaguChek Pro II	CoaguChekXsPlus	DLP_CoaguChekXsSeries.exe	6.4.80.39284
Roche Diagnostics Cobas h232 (old & new version)	CobasH232	DLP_CoaguChekXsSeries.exe	6.4.80.39284
Roche Diagnostics Cobas Liat	CobasLiat	DLP_CobasLiat.exe	6.5.27.43462
Roche Diagnostics Urisys 1100 (FW version 5.x+)	Urisys_1100_FW5.x	DLP_Urisys_1100_FW5.x.exe	6.4.98.41037
Siemens CLINITEK Advantus	Clinitек Advantus	DLP_Clinitек Advantus.exe	6.4.101.41037
Siemens CLINITEK Status Plus	ClinitекPlus	DLP_ClinitекStatusPlus.exe	6.5.22.42770
Siemens DCA Vantage	DCAvantage	DLP_DCAvantage.exe	6.5.22.42770
Siemens epoc Reader and Host	Epoc	DLP_Epoc.exe	6.5.40.44136
Siemens RAPIDPoint 500 and 500e (includes v5.1)	RapidPoint500/RapidPoint500e	DLP_RapidLab.exe	6.5.57.44251
Sight Olo	SightOLO	DLP_OLO.exe	6.5.11.42744
Sysmex POCH-100i	XP100_H	DLP_XP300_H.exe	6.4.33.16746

¹ **DLP Version:** DLP version installed by this release of the dNote Driver Pack.

Device Type	Driver Pack Install Name	Meditrac DLP Name	DLP Version ¹
TEM International Rotem Delta Gamma	RotemDelta	DLP Rotem Delta.exe	6.4.106.41037
Veda.Lab Easy Reader Plus	EasyReader	DLP EasyReaderPlusAstml.exe	6.4.56.41037
Werfen (IL) Avoximeter 1000E	Avox1000e	DLP Avox1000e.exe	6.4.40.41454
Werfen (IL) Avoximeter 4000	Avox4000	DLP Avox4000.exe	6.4.86.39799
Werfen (IL) GEM 4000 HL7 Direct to Device	Gem4000	DLP Gem4000 HL7.exe	6.4.109.41037
Werfen (IL) GEM 5000	Gem5000	DLP Gem4000 HL7.exe	6.4.109.41037
Werfen (IL) Hemochron Signature Elite	HemochronElite	DLP SignatureElite.exe	6.5.29.43150

¹ **DLP Version:** DLP version installed by this release of the dNote Driver Pack.